

Anti-Nuclear Antibody Screen (ANA) with Reflex to Titer

Order Name: **ANA SCR**

Test Number: 5500050

Revision Date: 11/12/2020

TEST NAME	METHODOLOGY	LOINC CODE
Anti-Nuclear Antibody Screen (ANA) with Reflex to Titer	Indirect Fluorescent Antibody	8061-4

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL	Serum	Clot Activator (SST or Red No-Gel)	Refrigerated
Instructions	Allow specimen to clot completely at room temperature. <b>Separate serum from cells ASAP or within 2 hours of collection.</b> Stability After separation from cells: Ambient 8 hours, Refrigerated 7 days, Frozen 3 months (avoid repeated freeze/thaw cycles).			

GENERAL INFORMATION

Testing Schedule	Mon - Fri																												
Expected TAT	3 Days																												
Clinical Use	To aid in the diagnose autoimmune diseases.																												
Notes	<p>If ANA screen is positive &gt;1:80; titer is automatically performed. ANA screening is done by IFA using HEp-2 Cells.</p> <p>See <a href="https://www.anapatterns.org/index.php">https://www.anapatterns.org/index.php</a> for current clinical associations and identified patterns.</p>																												
<table><tr><th>REFLEX ORDER CODE</th><th>REFLEX TEST NAME</th><th>RESULT CODE</th><th>RESULT LONG NAME</th></tr><tr><td>5512450</td><td>Antinuclear Antibody Titer</td><td>5500100</td><td>Antinuclear Antibody Titer</td></tr><tr><td></td><td></td><td>5500150</td><td>Antinuclear Antibody Pattern</td></tr><tr><td></td><td></td><td>5577550</td><td>Antinuclear Antibody Titer 2</td></tr><tr><td></td><td></td><td>5577650</td><td>Antinuclear Antibody Pattern 2</td></tr><tr><td></td><td></td><td>5577600</td><td>Antinuclear Antibody Titer 3</td></tr><tr><td></td><td></td><td>5577700</td><td>Antinuclear Antibody Pattern 3</td></tr></table>		REFLEX ORDER CODE	REFLEX TEST NAME	RESULT CODE	RESULT LONG NAME	5512450	Antinuclear Antibody Titer	5500100	Antinuclear Antibody Titer			5500150	Antinuclear Antibody Pattern			5577550	Antinuclear Antibody Titer 2			5577650	Antinuclear Antibody Pattern 2			5577600	Antinuclear Antibody Titer 3			5577700	Antinuclear Antibody Pattern 3
REFLEX ORDER CODE	REFLEX TEST NAME	RESULT CODE	RESULT LONG NAME																										
5512450	Antinuclear Antibody Titer	5500100	Antinuclear Antibody Titer																										
		5500150	Antinuclear Antibody Pattern																										
		5577550	Antinuclear Antibody Titer 2																										
		5577650	Antinuclear Antibody Pattern 2																										
		5577600	Antinuclear Antibody Titer 3																										
		5577700	Antinuclear Antibody Pattern 3																										
CPT Code(s)	86038 (86039 will bill in addition if reflexed to titer)																												
Lab Section	Immunology - Serology																												