## Labcorp Oklahoma, Inc. Test Directory

## **Umbilical Cord Toxicology**

Six inches of Umbilical Cord

Mass Spectrophotometry

Methodology

Order Name:

Revision Date: 01/13/2014

**TEST NAME METHODOLOGY LOINC CODE Umbilical Cord Toxicology SPECIMEN REQUIREMENTS** Specimen Specimen Volume (min) Specimen Type Specimen Container Transport Environment Instructions **Collection Instructions:** 1. Immediately cut six inches of the umbilical cord. 2. Wash with saline and place in designated sterile container. 3. Label container appropriately with patient identification, date of time of collection, & collector's initials. Complete Chain of Custody form. Container: Sterile container with red lid designating for toxicology. Storage and Transport: Room temperature Stability: Good for 3 weeks refrigerated Rejection Due To: No chain of custody form **Minimum Volume:** 

GENERAL INFORMATION	
Testing Schedule	Monday - Saturday
Expected TAT	3 Days
Lab Section	United States Drug Testing Laboratories, Inc.

Service provided by Labcorp Oklahoma, Inc. All Rights Reserved. © 2003 - 2024