## Labcorp Oklahoma, Inc. Test Directory

## **C-Peptide**

Order Name: CPR Revision Date: 12/03/2019

TEST NAME			METHODOLOGY	LOINC CODE
C-Peptide				
SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	0.5mL (0.4 mL)	Serum	Red Top	Frozen
Instructions	<ul> <li>Patient Preparation:</li> <li>1. Patient should fast for 8 hours.</li> <li>2. For 12 hours before this test do not take multivitamins or dietary supplements containing biotin (vitamin B7), which is commonly found in hair, skin, and nail supplements and multivitamins.</li> </ul>			
	Label specimen with Mobilab label or with patient name (first and last), date of birth, date and time of collection, collector initials, and test(s) being ordered.			
Reference Range	1.1-4.4 ng/mL			
	Reference intervals have not been formally verified in-house for pediatric patients. The published literature indicates that reference intervals for adult and pediatric patients are comparable.			
Methodology	Electrochemiluminescence Immunoassay (ECLIA)			
GENERAL INFORMAT	Monday through Friday 5 a.m12 a.m.; Saturday 6 a.m6 p.m.			
Expected TAT	2-3 Days			
Notes	Specimen Stability: Frozen (preferred)- 30 Days Refrigerated - 7 Days			
CPT Code(s)	84681			
Lab Section	NRLS-Mayo Medical Laboratories			