

ADAMTS13 Activity Reflex Panel

Order Name: **ADAMTS13 Reflex**

Test Number: 5194453

Revision Date: 12/12/2022

TEST NAME	METHODOLOGY	LOINC CODE
ADAMTS13 Activity Reflex Panel		

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL (0.3 mL)	Platelet Poor Plasma	Sodium Citrate 3.2% (Blue Top)	Frozen
Instructions	<p><b>Notes:</b> 0.3 mL (Note: This volume Does not allow for repeat testing)</p> <p><b>Specimen Type:</b> Blue-top (sodium citrate) tube</p> <p><b>Specimen Storage:</b> <b>FREEZE</b></p> <p><b>Specimen Collection:</b> Blood should be collected in a blue-top tube containing 3.2% buffered sodium citrate. Evacuated collection tubes must be filled to completion to ensure a proper blood to anticoagulant ratio. The sample should be mixed immediately by gentle inversion at least six times to ensure adequate mixing of the anticoagulant with the blood. Centrifuge for 10 minutes and (using a transfer pipette) carefully remove two-thirds of the plasma without disturbing the cells. Deliver to a plastic transport tube, cap, and recentrifuge for 10 minutes. Use a second plastic pipette to remove the plasma, staying clear of the platelets at the bottom of the tube. Transfer the plasma into a Sterile Pour Off Tube or LabCorp PP transpak frozen purple tube with screw cap (LabCorp ID 49482). <b>The specimen should be FROZEN immediately and maintained frozen until tested.</b> To avoid delays in turnaround time when requesting multiple tests on frozen samples, PLEASE SUBMIT SEPARATE FROZEN SPECIMENS FOR EACH TEST REQUESTED.</p> <p><b>Special Instructions:</b> ADAMTS13 Activity with reflex to ADAMTS13 Antibody when Activity is less than 30%.</p> <p><i>Collection Notes: A discard tube is not required prior to collection of coagulation samples. When noncitrate tubes are collected for other tests, collect sterile and nonadditive (red-top) tubes prior to citrate (blue-top) tubes. Any tube containing an alternate anticoagulant should be collected after the blue- top tubes. Serum gel tubes and serum tubes with clot initiators should also be collected after the citrate tubes.</i></p>			

GENERAL INFORMATION	
Expected TAT	2 - 4 days
Performing Labcorp Test Code	117921
Notes	Labcorp Test Code: 117921
CPT Code(s)	<p>ADAMTS13 Activity: 85397</p> <p>Possible Reflex:</p> <p>Order Code: <b>(5194454)</b> .Reflex ADAMTS13 Antibody</p> <p>Result Code: <b>(5194456)</b> ADAMTS13 Antibody</p> <p>Result Code: <b>(5194457)</b> ADAMTS13 Comment</p> <p>CPT: 83520</p> <p>Possible Reflex:</p> <p>Order Code: <b>(5194455)</b> .Reflex ADAMTS13 Comment</p> <p>Result Code: <b>(5194457)</b> ADAMTS13 Comment</p> <p>CPT: n/a</p>
Lab Section	Reference Lab