

## Trichomonas Antigen

Order Name: **TRICH AG**  
Test Number: 2915460  
Revision Date: 10/01/2022

TEST NAME	METHODOLOGY	LOINC CODE
Trichomonas Antigen	Enzyme Immunoassay	31978-0

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	See Instructions	Genital-Vaginal	Copan eSwab - White (Regular size)	Refrigerated
Alternate 1	See Instructions	Genital-Vaginal	Copan eSwab - Green (Mini-tip)	Refrigerated
Alternate 2	See Instructions	Genital-Vaginal	Wet prep saline	Refrigerated
Instructions	Copan eSwab - White, or Saline from a Wet Prep tube. Keep Refrigerated (4-5°C) Stability: Room Temperature: 24hrs, Refrigerated: 36hrs, Frozen (-20°C): 36hrs. Samples contaminated with preparations containing iodine or by the immediate use of vaginal lubricants are not recommended.			

GENERAL INFORMATION	
Testing Schedule	1-2 Days
Expected TAT	Sun-Sat
Notes	Trichomonas vaginalis antigen testing using an eSwab collection container was developed and its performance characteristics determined by Labcorp Oklahoma, Inc. It has not been cleared or approved by the FDA. The laboratory is regulated under CLIA as qualified to perform high-complexity testing. This test is used for clinical purposes. It should not be regarded as investigational or for research.
CPT Code(s)	87808
Lab Section	Microbiology