

## Bilirubin, Body Fluid

Order Name: **BFTBILI**  
Revision Date: 01/12/2013

TEST NAME	METHODOLOGY	LOINC CODE
Bilirubin, Body Fluid		

### SPECIMEN REQUIREMENTS

Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	2.0 mL (1 mL)	Body Fluid	Serum Gel Tube	See Instructions

<b>Instructions</b>	Place body fluid in serum gel vacutainer tube(s). Spin down and send 2 ml. (minimum volume: 1 mL) of fluid. <b>Keep refrigerated. Avoid hemolysis. Protect from light.</b>			
	Note:			
	1. Indicate specimen source on request form.			
	2. Label specimen with patients name( first and last) or other unique identifier, patient hospital identification number or Social Security Number, date and time of draw, collector initials, and test(s) being ordered.			

<b>Reference Range</b>	An interpretative report will be provided.			
------------------------	--	--	--	--

<b>Methodology</b>	Includes bilirubin indirect			
--------------------	-----------------------------	--	--	--

### GENERAL INFORMATION

<b>Testing Schedule</b>	Monday through Saturday TAT: 2 hours STAT TAT: 1 hour
<b>CPT Code(s)</b>	82247 - bilirubin, total 82248 - bilirubin, direct
<b>Lab Section</b>	NRLS-Core Chemistry