Labcorp Oklahoma, Inc. Test Directory

Lactate Dehydrogenase (LDH)

Order Name: LDH

Revision Date: 01/12/2013

TEST NAME	METHODOLOGY	LOINC CODE
Lactate Debudrageness (LDH)		

Lactate Dehydrogenase (LDH)

SPECIMEN REQUIREMENTS					
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment	
Alternate 1	1.0 mL (0.5 mL)	Serum	Serum Gel Tube	Room Temperature	
Alternate 2	1.0 mL (0.5 mL)	Plasma	Lithium Heparin	Room Temperature	
Instructions	Draw blood in a serum gel tube(s). Centrifuge clotted specimen within 2 hours of collection perferred. Send 1 mL (minimum volume: 0.5 mL) of serum refrigerated. Avoid hemolysis. Note: Label specimen with patient name (first and last) or other unique identifier, patient hospital identification number or Social Security number, date and time of draw, collector initials, and test(s) being ordered.				
Reference Range	See Interpretive Report				
Methodology	Enzymatic				

GENERAL INFORMATION		
Testing Schedule	Monday through Sunday	
Expected TAT	3 hours upon receipt in laboratory	
Stat TAT	45 minutes upon receipt in laboratory	
CPT Code(s)	83615	
Lab Section	NRLS-Core Chemistry	

Service provided by Labcorp Oklahoma, Inc. All Rights Reserved. © 2003 - 2025