

Stool for Cells, Wright Stain

Order Name: **STOOLCEL**  
Revision Date: 03/22/2013

TEST NAME	METHODOLOGY			LOINC CODE
Stool for Cells, Wright Stain				
SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Instructions	Collect fresh stool (fecal) specimen in a wide-mouth container, bedpan, or plastic wrap loosely covering toilet and held in place with toilet seat. Collect specimen on child in diapers by placing pedi-urine bag over rectum.			
	Place stool into sterile container; label container with first and last name, date of birth, date and time of collection, and collector's initials; <b>Keep refrigerated and transport to laboratory within 2 hours.</b>			
Reference Range	No WBC Seen			
Methodology	Wright Stain			

GENERAL INFORMATION	
Testing Schedule	Performed daily
Expected TAT	24 hours
Stat TAT	1 hour
Notes	Place stool into sterile container; label container with first and last name, date of birth, date and time of collection, and collector's initials;  <b>Keep refrigerated and transport to laboratory within 2 hours.</b>
CPT Code(s)	87205
Lab Section	NRLS-Core Hematopathology