

Infliximab Activity and Neutralizing Antibody

Order Name: **Infliximab NAB**
Test Number: 6905051
Revision Date: 01/02/2020

TEST NAME	METHODOLOGY	LOINC CODE
Infliximab Activity and Neutralizing Antibody	Electrochemiluminescent Immunoassay	

SPECIMEN REQUIREMENTS

Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	2 mL (1 mL)	Serum	Clot Activator SST	Frozen

Instructions

Notes: 1 mL (Note: This volume Does NOT allow for repeat testing).
Specimen Type: Red-top tube or gel-barrier tube
Specimen Storage: Frozen (preferred) or refrigerated Stability: Room temperature: 7 days Refrigerated: 7 days Frozen: 359 days Freeze/thaw cycle: stable x6
Specimen Collection: Serum must be separated from cells within 45 minutes of venipuncture. Send serum in a plastic transport tube. To avoid delays in turnaround time when requesting multiple test on frozen samples, PLEASE SUBMIT SEPARATE FROZEN SPECIMENS FOR EACH TEST REQUESTED.
Special Instructions: Testing is performed at Esoterix Endocrinology Laboratory: ES#803870 Test includes Serial Monitoring.

GENERAL INFORMATION

Expected TAT	5-10 days
Performing Labcorp Test Code	503870
Notes	Labcorp Test Code: 503870
CPT Code(s)	80230, 82397
Lab Section	Reference Lab