

## Thyroglobulin Panel

Order Name: **Thyroglob Pnl**

Test Number: 3612483

Revision Date: 12/22/2023

TEST NAME	METHODOLOGY	LOINC CODE
<b>Thyroglobulin Autoantibody (TG Ab)</b>	Enzyme Immunoassay	8098-6
<b>Thyroglobulin Level</b>	Immunoassay (IA)	3013-0

### SPECIMEN REQUIREMENTS

Specimen	Specimen Volume (mL)	Specimen Type	Specimen Container	Transport Environment
<b>Preferred</b>	<b>3 mL (1.8 mL)</b>	<b>Serum</b>	<b>Clot Activator SST</b>	<b>Frozen</b>

**Instructions**

Preferred is Two Aliquots of Serum:  
(1) Allow specimen to clot fully, centrifuge, remove 2mL of Serum and Freeze.  
(2) 1 mL Serum kept Refrigerated.  
Serum stability: Ambient 8 hours, Refrigerated 3 days, Frozen 2 months

*Thyroglobulin by immunoassay is highly susceptible to biotin interference. Patients taking biotin supplements – particularly at high doses – may show falsely low or undetectable thyroglobulin levels. Please consult with lab if biotin interference is suspected.*

### GENERAL INFORMATION

<b>Testing Schedule</b>	Mon-Fri
<b>Expected TAT</b>	1-4 days from set up
<b>CPT Code(s)</b>	86800, 84432
<b>Lab Section</b>	Chemistry