

Flunitrazepam and Metabolites, Quantitative, Serum

Order Name: **FLUNTR SCR**

Test Number: 4013050

Revision Date: 01/04/2023

| TEST NAME | METHODOLOGY | LOINC CODE |
|--|--|------------|
| Flunitrazepam and Metabolites, Quantitative, Serum | Liquid Chromatography/Tandem Mass Spectrometry | |

| SPECIMEN REQUIREMENTS | | | | |
|-----------------------|--|---------------|----------------------------------|-----------------------|
| Specimen | Specimen Volume (mL) | Specimen Type | Specimen Container | Transport Environment |
| Preferred | 5 mL (2 mL) | Serum | Clot Activator (Red Top, No-Gel) | Refrigerated |
| Instructions | <p>Specimen Type: Red-top tube OR green-top (heparin) tube. Gel-barrier tubes are NOT recommended.</p> <p>Specimen Storage: Room temperature is adequate for shipping and/or short-term storage (three days); specimens may also be refrigerated or frozen.</p> <p>Specimen Collection: Serum or plasma should be separated from cells within two hours of venipuncture. Submit serum or plasma in a plastic transport tube.</p> <p>Special Instructions: Testing referred to MedTox Laboratories MX#438</p> | | | |

| GENERAL INFORMATION | |
|------------------------------|---------------------------|
| Expected TAT | 10-14 days |
| Performing Labcorp Test Code | 808443 |
| Notes | Labcorp Test Code: 808443 |
| CPT Code(s) | 80346 (HCPCS: G0480) |
| Lab Section | Reference Lab |